




Downloading & Installing Libraries for 3shape

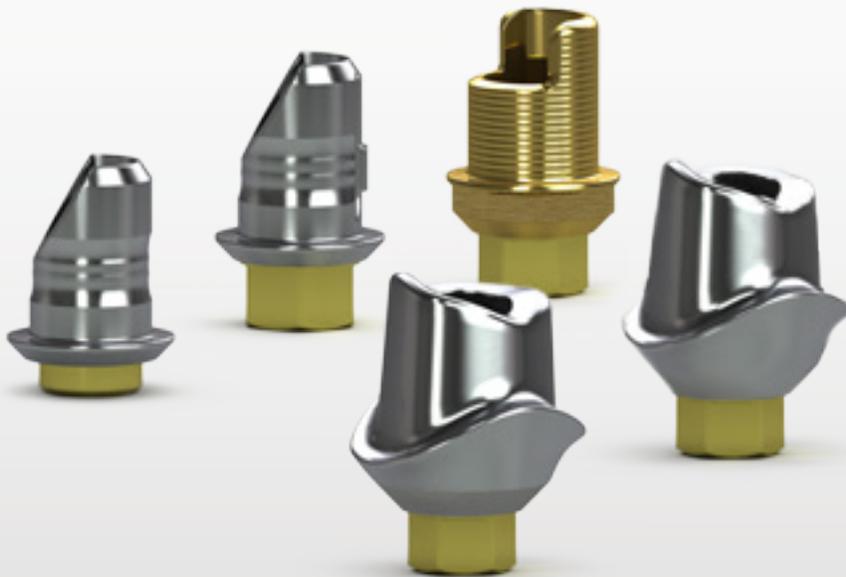

SOPHISTICATED MILLING SOLUTIONS


STEP 1: Download Files from Vulcan Website

Visit www.vulcandental.com and navigate the site to **Resources**>**Downloads** to access the libraries page. Click on the orange download link and the download will begin automatically. A ".zip" file will go to your downloads folder. Open this folder and drag the ".DME" file to your desktop.



STEP 2: Open 3Shape Dental System Control Panel

Click on the hex nut icon to open the Control Panel from Dental System. If this is the first time installing a Vulcan Library please skip to step 5. Otherwise, if you currently have a Vulcan Implant Library on your 3Shape system continue to the next page.

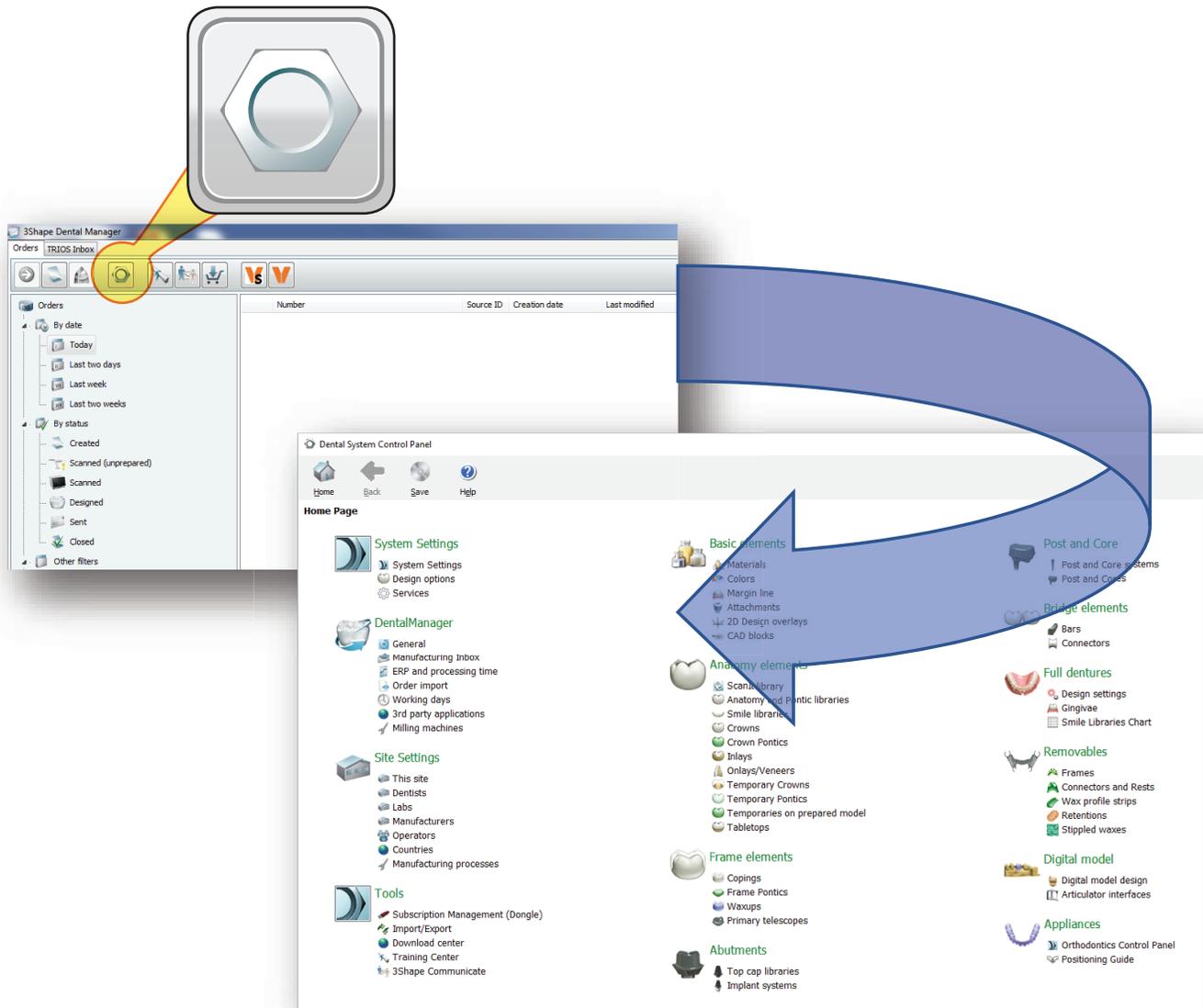


STEP 3: Remove Previous Vulcan Libraries

From the control panel, click on **Abutments>Implant Systems**. Inside, highlight the systems to remove and click delete. Then click Empty the Recycle Bin.

The image shows a screenshot of the 'Abutments' control panel. A callout box on the left highlights the 'Implant systems' option under the 'Abutments' menu. The main window displays the 'Implant systems' section with a list of implant systems, including '3Shape Implant 4.3 mm', 'BioHorizons Ti (MA2.0)', 'BioHorizons Ti Laser-Lok (MA2.0)', 'BioHorizons Hybrid (MA2.0)', 'BioHorizons Hybrid Laser-Lok (...)', 'BioHorizons Hybrid Non-Hexed ...', 'BioHorizons Multi-Unit Library', and 'iSy Titanium-base CAD-CAM cr...'. Below the list are buttons for 'Add', 'Copy', 'Delete', 'Move up', and 'Move down'. A blue arrow points to the 'Delete' button. The 'Recycle bin' section at the bottom shows 'No items', 'Restore all', and 'Empty the Recycle Bin'. A blue arrow points to the 'Empty the Recycle Bin' button. The right side of the window shows the 'Edit Implant system' section with 'Implant system categories' including '3Shape Demo', 'Vulcan Metal Analog...', 'Multi-Unit Beta', 'iSy incl. P3010.7012', 'DEDICAM one-piece for...', and 'DEDICAM qinqiva forme...'. Below this are buttons for 'Add', 'Rename', and 'Delete'.

STEP 4: Remove Previous Vulcan Materials

You will find Vulcan Materials and setting in the following areas: **Manufacturing processes, Materials, Crowns, Crown Pontics, Inlays, Onlays/ Veneers, Tabletops, Copings, Frame Pontics, Waxups, and Digital model design.** All settings will be labeled as Vulcan and must be deleted individually. After deleting please empty Recycle Bin.

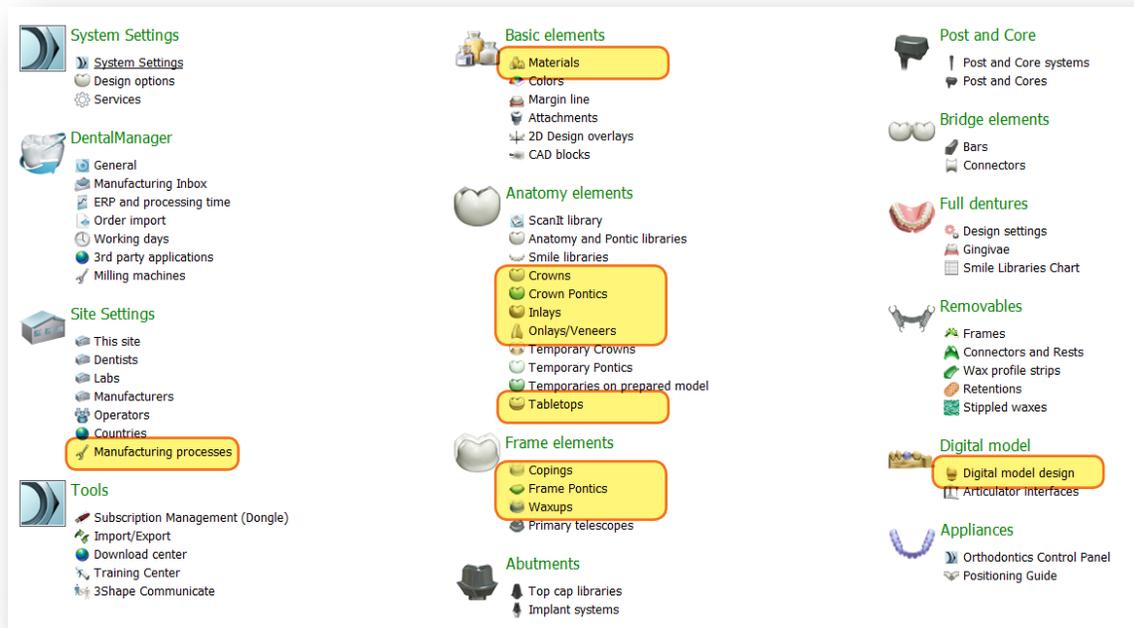

Submenu Examples

Anatomy elements > Crowns

+ Add + Copy - To Recycle Bin

Name	ID
Vulcan Zr Crown	86902_Vulcan Zirconia
Vulcan Zr Crown on Ti Abut...	86902_Vulcan Zirconia on T...
Vulcan Temp Crown	86902_Vulcan Temp Crow...
Vulcan Temp Crown on Ti ...	86902_Vulcan Temp Crow...

Anatomy elements > Inlays

Inlay

- Vulcan Zr Inlay
- Vulcan Temp Inlay

+ Add
+ Copy
- Delete
Move up
Move down

Recycle bin

- No items
- Restore all
- Empty the Recycle Bin

Details


Total: 2 items

STEP 5: Import New Vulcan Library

From the control panel, navigate to **Tools>Import/Export**. Once at the Import/Export window, click **Import Materials** and browse for the .DME file on your desktop. Follow the on-screen prompts and use RECOMMENDED settings.

Exit the control panel and click Yes when prompted to save changes before closing.

The Vulcan library has now been successfully installed. You must restart dental system before changes will take effect.

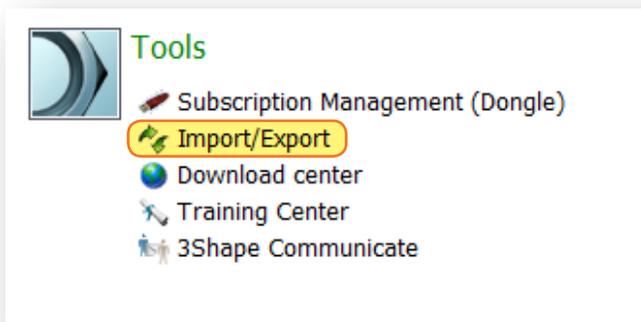

2300 Riverchase Center, Suite 825
Birmingham, AL 35244
Toll free: 1.844.484.2301 • Direct: 205.484.2301
www.vulcandental.com